

Züm SP Travel Kit Packing Instructions



SCAN QR CODE TO WATCH
INSTRUCTION VIDEO ON
YOUTUBE



1. Pack seat into seat bag with seat support bar facing the opening of the bag.



2. Pack casters, backrest tubes, armrests, cane handles, footrests, anti tippers and wheel locks as shown above.



3. Open suitcase and the foam insert should be positioned as shown on the right half of the suitcase.

SUITCASE



4. Place fully collapsed Züm frame on top of the foam insert. Orient the frame as shown in the image above.



5. Pull up on the Züm frame to open and pack the seat bag between the side frames.



6. Place the parts bag on top of the Züm frame ensuring all components in the bag are as flat as possible.



7. Place the molded upholstery on top of the parts bag.



8. Fasten the strap across the components.



9. Fold the left half of the case onto the right half and zip the two halves together.

WHEEL BAG



10. Pack SP wheels and quick-release axles into individual pockets.



11. Zip all pockets and fold handle cover over Velcro to lock.